
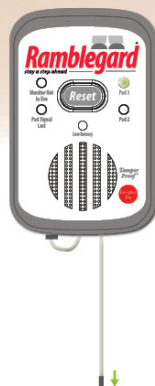
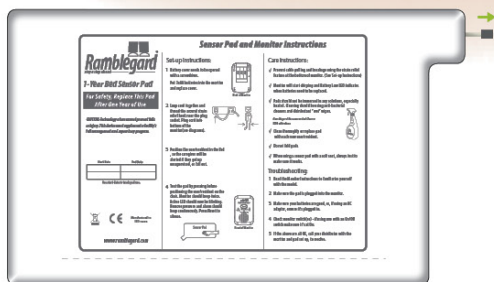


Code: BG1 - Bedgard

quick start instructions

Ramblegard

stay a step ahead

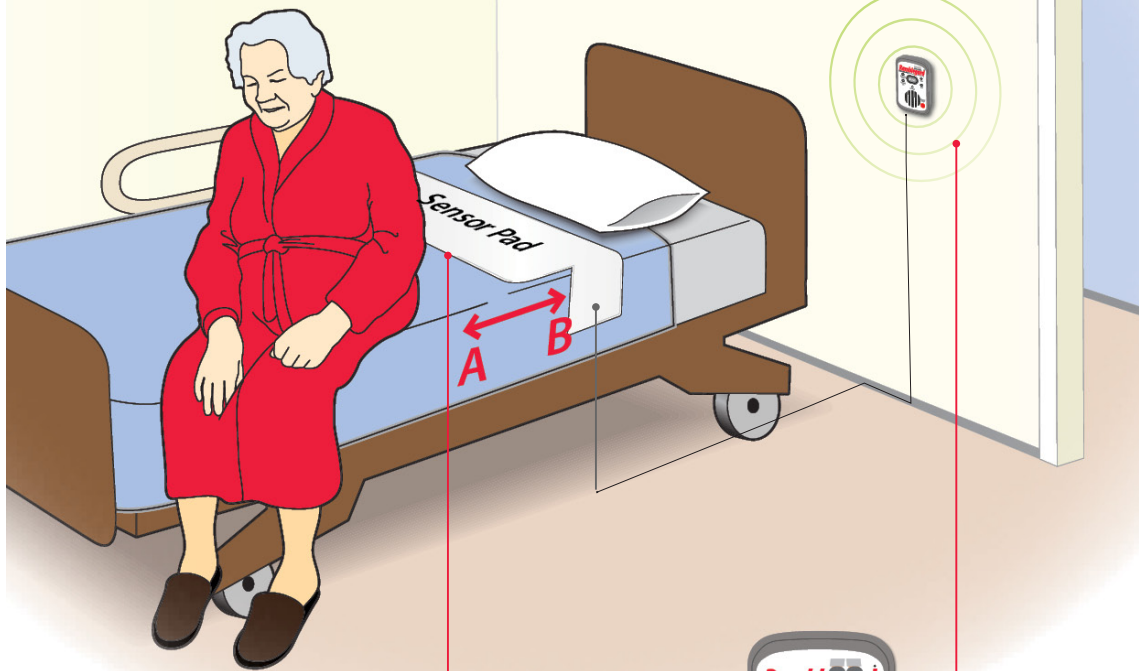

1. Remove the monitor from the protective holder.
2. Open the Battery Compartment and insert 3x AA Batteries as illustrated overleaf
3. Connect monitor to Sensor Mat as illustrated (when the monitor stops beeping and the lights go out, the system is ready for testing.)
4. Position the Sensor Pad on the bed as illustrated.
Position A - To alarm when patient gets out of bed Position B - To alarm when the patient sit up in bed
5. Situate the Ramblegard Monitor in the most accessible position for the Carer.
6. TEST the system by pressing down on the pad. The LED should flash several times, and the monitor should beep twice to indicate "system in use". Release pressure from the pad and the monitor should alarm. Press the reset button to silence.
7. To reset the system either:
 - Press the reset button (or use optional carer key) to silence and set the system on standby.
 - or reapply pressure to the pad to silence and rearm the system.

further details overleaf

Product Information


Product Code: BG1



Product Name
Wired Bedgard

System Contents

1x Ramblegard Wired Monitor - RG1
1x Wired Bed Sensor Mat - RG12

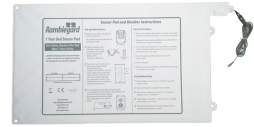

Ramblegard have programmed the components included as a plug and play system.

If individual components need reprogrammed or replaced please refer to the relevant specification sheets available to download at www.ramblegard.com

QUICKSTART INSTRUCTIONS see overleaf

Carers should also consider the following points when determining the most appropriate position for the the Ramblegard Sensor Pads & Monitors in care environments:


Bed Sensor Pad Placement

- Position pad under the shoulders to alert when the patient sits up in bed.
- Position pad under the hips/buttocks to alert when the patient vacates the bed.

Monitor Placement - Attach using the quick release strap supplied or attach to wall or vertical surface using integral keyhole slots.

- The monitor can be placed within range of the connecting cable. -extension cables available on request
- The monitor should be placed where it cannot be tampered with by patients especially in dementia & challenging environment situations.

Back of Monitor

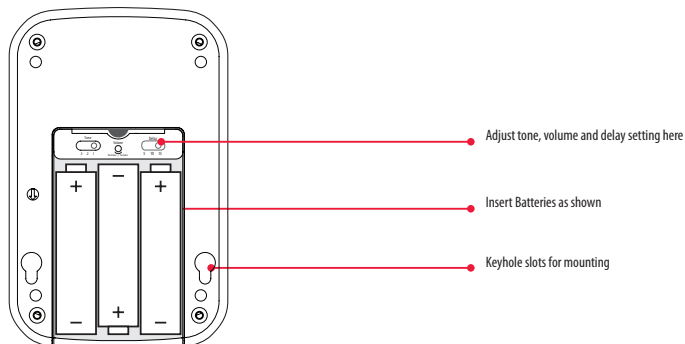

Disclaimer: Whilst every effort has been made to ensure the safety of this product, we cannot be held responsible for any damage or injury sustained when the equipment is in use. It is the responsibility of the user to install the system and ensure it is tested and operating correctly. The system is not designed to replace good care giving practice.