


Code: BG1-NC – Bedgard Nurse Call
quick start instructions

Ramblegard
 stay a step ahead

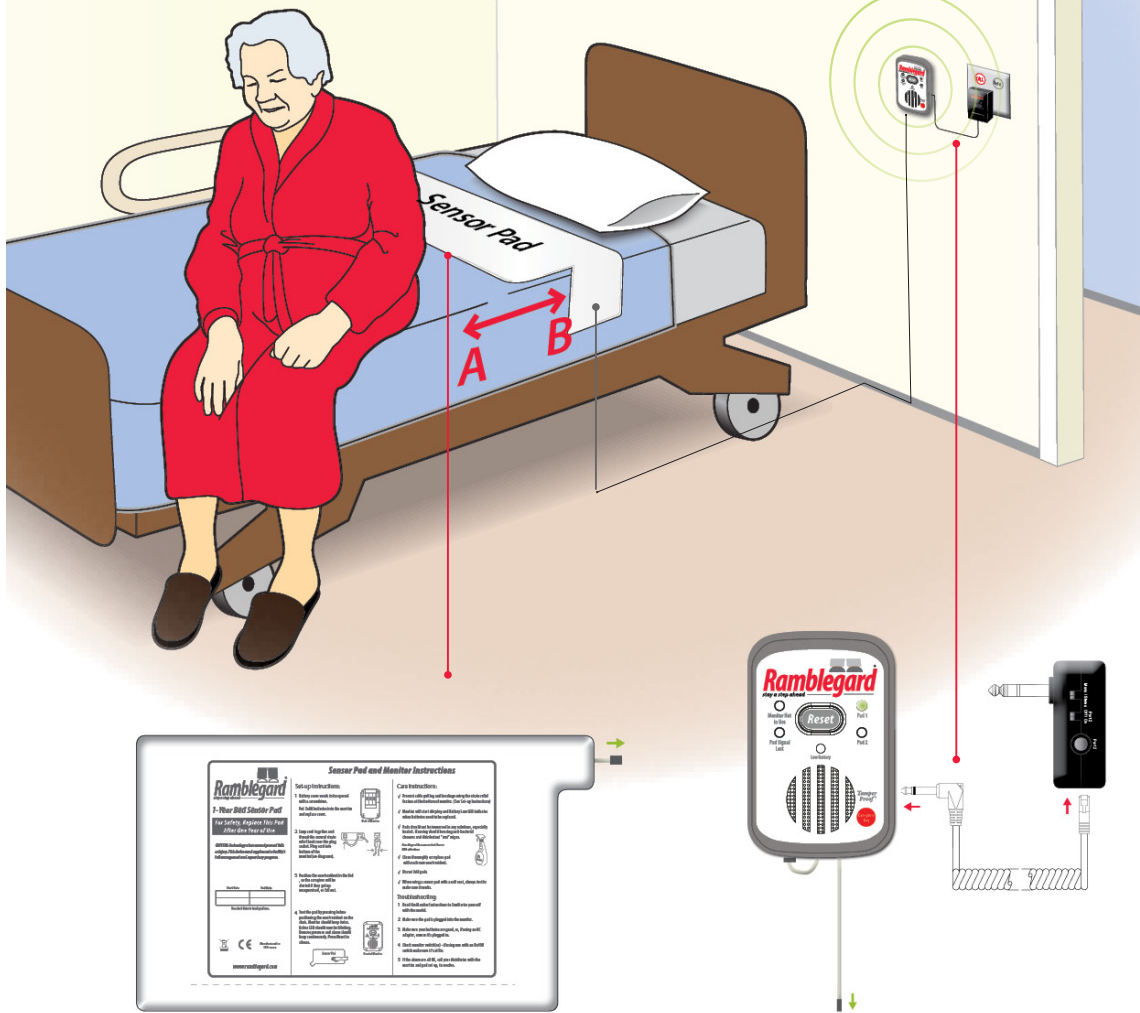

further details overleaf

1. Remove the monitor from the protective holder.
2. Open the Battery Compartment and insert 3x AA Batteries as illustrated overleaf
3. Connect monitor to Sensor Mat as illustrated (when the monitor stops beeping and the lights go out, the system is ready for testing.)
4. Position the Sensor Pad on the bed as illustrated.
 Position A – To alarm when patient gets out of bed Position B – To alarm when the patient sit up in bed
5. Situate the Ramblegard Monitor adjacent to the nurse call socket and connect as illustrated.
6. TEST the system by pressing down on the pad. The LED should flash several times, and the monitor should beep twice to indicate “system in use”. Release pressure from the pad and the monitor should alarm. Press the reset button to silence.
7. To reset the system either:
 - First Press the reset button on the monitor (or use optional carer key) and then refer to the correct Nurse-Call reset point to silence and set the system on standby.
 - or reapply pressure to the pad and then refer to the correct Nurse-Call reset point to silence and rearm the system.

Product Information

Product Code: BG1-NC



Product Name

Wired Bedgard - Nurse Call

System Contents

- 1x Ramblegard Corded Monitor - RG1
- 1x Ramblegard Corded Bedmat - RG12
- 1x Jack Adaptor - RG4
- 1x Monitor to Nursecall Lead - RG6



Ramblegard have programmed the components included as a plug and play system.

If individual components need reprogrammed or replaced please refer to the relevant specification sheets available to download at www.ramblegard.com

QUICKSTART INSTRUCTIONS see overleaf

Carers should also consider the following points when determining the most appropriate position for the the Ramblegard Sensor Pads & Monitors in care environments:


Bed Sensor Pad Placement

- Position pad under the shoulders to alert when the patient sits up in bed.
- Position pad under the hips/buttocks to alert when the patient vacates the bed..

Monitor Placement - Attach using the quick release strap supplied or attach to wall or vertical surface using integral keyhole slots.

- The monitor must be placed close to a nurse call point and within range of the sensor connecting cable. - extension cables available on request
- The monitor should be placed where it cannot be tampered with by patients especially in dementia & challenging environment situations

Back of Monitor

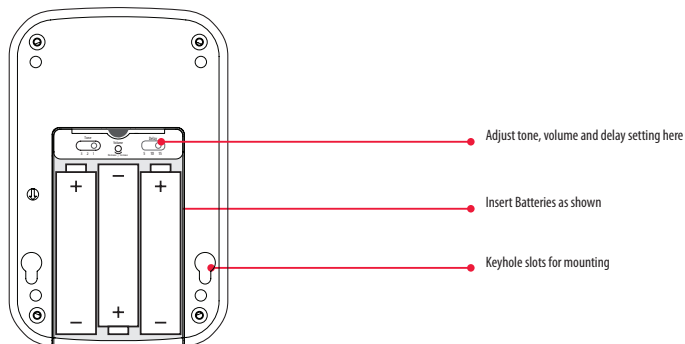

Disclaimer: Whilst every effort has been made to ensure the safety of this product, we cannot be held responsible for any damage or injury sustained when the equipment is in use. It is the responsibility of the user to install the system and ensure it is tested and operating correctly. The system is not designed to replace good care giving practice.