

10 POINT TEST BEFORE USE METHOD

In order to keep your Ramblegard Falls Alarm in good working order it will need to be regularly inspected and maintained. Whilst every effort has been made to ensure the safety and reliability of this product, we cannot be held responsible for any damage or injury sustained as a result of misuse of the equipment or negligence by the carer. With this in mind, it is imperative that you test your Ramblegard system before every use.

Key Points to remember when setting up a pad with a patient:

- 1** If your monitor/pager has an ON/OFF switch, ensure it is switched ON.
- 2** If there is an audible/visual 'Low Battery' alert, replace the batteries immediately.
- 3** If it is a Cordless system, ensure the transmitter in the sensor pad is switched to the ON position.
- 4** Check the start date on the pad. If the pad is over 12 months old, Ramblegard recommend you take it out of service and seek a replacement.
- 5** Ensure that the sensor pad is placed on a smooth flat surface and the pad is not creased or folded in any way.
- 6** Check that you have positioned your sensor pad correctly.
- 7** Ensure the matching monitor and sensor pad(s) are used together.
- 8** We suggest that your sets are clearly numbered to avoid confusion.
- 9** To test, activate the sensor to confirm that it is alarming and resetting correctly.
- 10** When you are satisfied with the points above please complete the daily checks sheet provided before putting the system into use.