


BOND LITE SET-UP INSTRUCTIONS




Follow the steps below to set-up your falls equipment:

1.

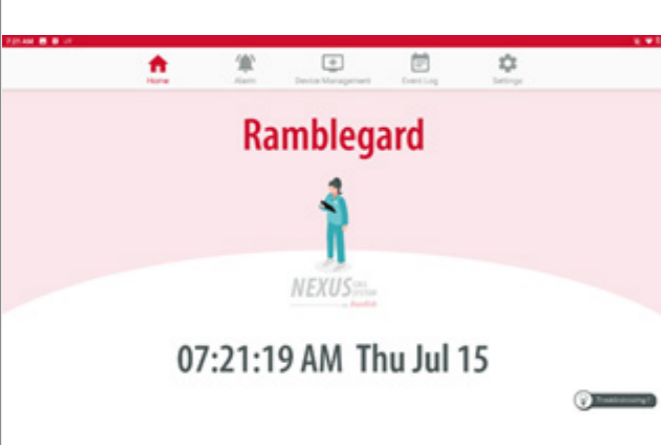


The screen should be illuminated and the display should look similar to this.

4.

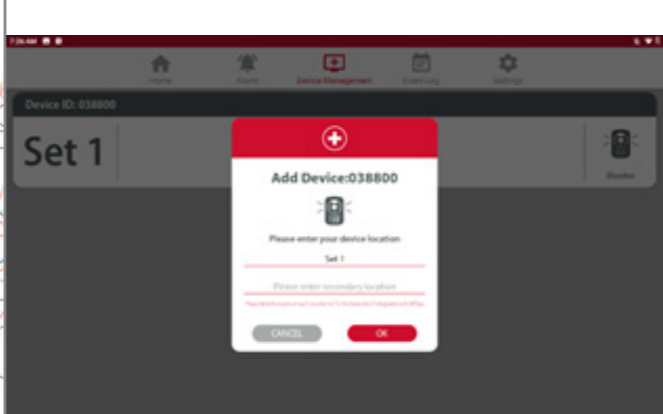


Assign sensor location. Find the set number you want to deploy and select it. The location box will appear. Change the secondary location to the room number where the alarm will be located.

2.

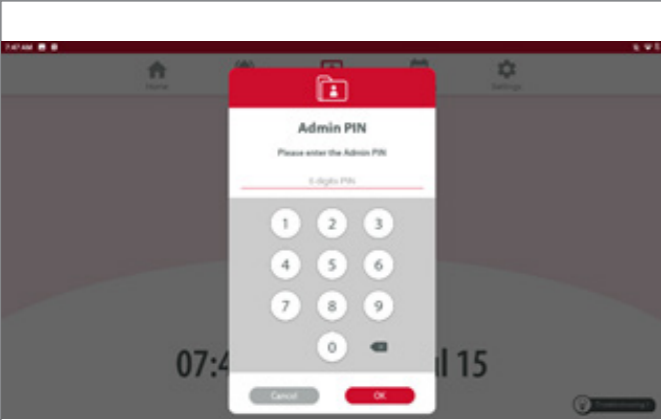


Tap 'DEVICE MANAGEMENT' from the list of options at the top of the screen, this will prompt a request for an admin pin.
Your admin pin is:
1 2 3 4 5 6

5.

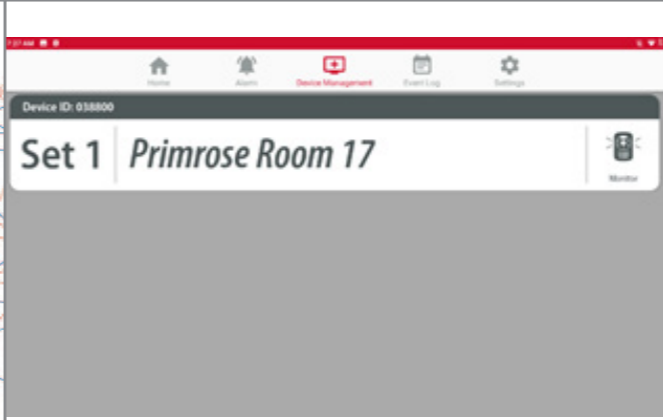


Confirm changes. Once you have selected the new location, you will be able to view this back in the device management screen.

3.

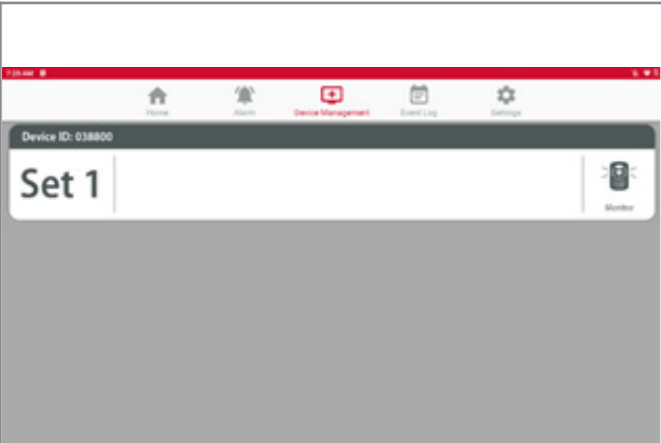

After inputting your admin pin, the device management screen will appear.
This screen will show all the sensors available to you, and from here, you can assign them to the appropriate location.

6.

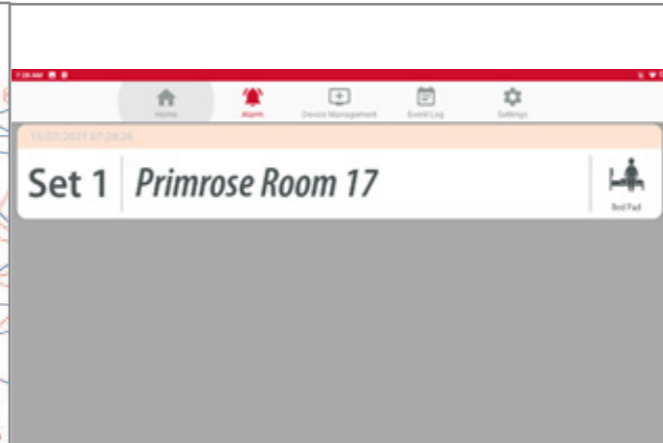


Tap 'Home' to return to the home screen.
Always test the sensor pads before use to ensure they are triggering alerts.
When an alert is triggered, the location will display on the central screen and on the pager devices.


BOND LITE SET-UP INSTRUCTIONS





Follow the steps below to set-up your falls equipment:

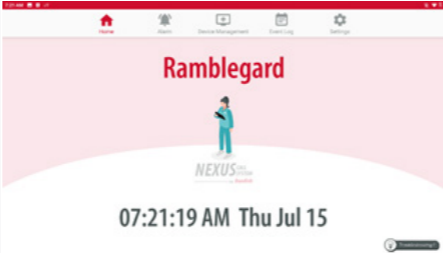
1.  Check that the 'Alarm Set' numbers on each item match

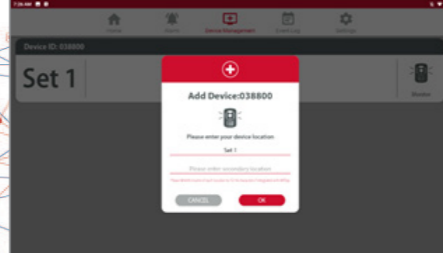4.  The screen should be illuminated and the display should look similar to this.


7.  Assign sensor location. Find the set number you want to deploy and select it. The location box will appear. Change the secondary location to the room number where the alarm will be located.


2.  Switch transmitter 'ON'
i) Take transmitter out of pocket on sensor pad.
ii) Using a pen, move switch to 'ON' position.
iii) Put transmitter back into pocket and seal shut.

5.  Tap 'DEVICE MANAGEMENT' from the list of options at the top of the screen, this will prompt a request for an admin pin.
Your admin pin is:
1 2 3 4 5 6

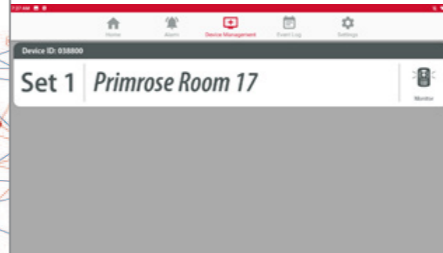8.  Confirm changes. Once you have selected the new location, you will be able to view this back in the device management screen.

3.  To deactivate SLEEP mode hold RESET button until LCD shows 'ON'

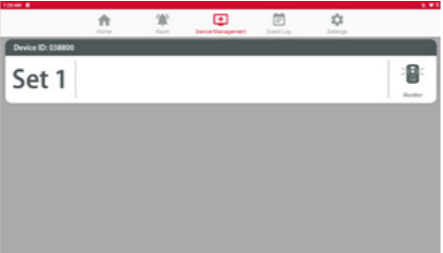6.  After inputting your admin pin, the device management screen will appear.
This screen will show all the sensors available to you, and from here, you can assign them to the appropriate location.

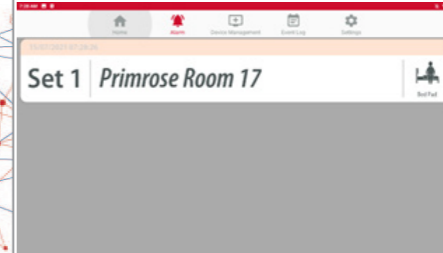9.  Tap 'Home' to return to the home screen.
Always test the sensor pads before use to ensure they are triggering alerts.
When an alert is triggered, the location will display on the central screen and on the pager devices.