

Quick Start Guide

Wired Companion Monitor Nurse-call Kit

Applies to product codes:

R02-0002 | R03-0002 | R04-0002 | R01-0011C | R01-0203C |

System Contents	
Monitor	RG1 Companion Monitor
Lead	3m breakaway or 8m breakaway
Sensor	RG15 Wired Standard Floorgard RG58 Wired Deluxe 90 Floorgard / RG37 Wired Deluxe Floorgard RG10 Wired Seatgard / RG48 Wired Seatgard RG12 Wired Bedgard / RG46 Wired Bedgard
Nurse-call Lead	RG4 jack and RG6 coiled lead / Or appropriate interface cable for your nurse-call system.

Sensor Placement

Bedgard: Place on top of the mattress and under the bed sheet. The pad can be placed anywhere between the shoulders to the hips depending on the condition of the patient and how early you want the alert. For further guidance on placement, please refer to the pad positioning flyer. Keep in mind the routing of the cable to the Monitor to prevent damage from moving parts of the bed frame.

Seatgard: Place mat on the relevant seat on top of or under a cushion, giving consideration to the position of the monitor and the least intrusive method of installing the equipment.

Floorgard: Place mat at or near to the area to be monitored such as a doorway or side of the bed.

Receiver/Monitor Placement

The monitor can be positioned on a wall via the plastic wall bracket or on a bed or chair via the hanging strap.

General Operation

The monitor triggers an alarm when the sensor mat is activated and can be reset at the monitor or by reactivating the pad.



Quick Start Guide

Wired Companion Monitor Nurse-call Kit

Follow the steps below to set up your equipment:

- 1.** Pull out the battery tab to power up the monitor or insert 3 AA batteries into the monitor. A long beep will confirm the batteries have been inserted correctly.
- 2.** Fit the rubber cover and mount the monitor in the desired location using the supplied hanging strap or wall bracket.
- 3.** Attach the supplied nurse-call interface cable between the monitor and nurse-call panel. If RG4 Jack is supplied, confirm mono/stereo switches are in the correct position.
- 4.** Ensure that the sensor pad is placed on a smooth flat surface and the pad is not creased or folded in any way.
- 5.** Check that you have positioned your sensor pad correctly and connect the cable with the monitor.
- 6.** To test, activate the sensor to confirm that it is alarming and resetting correctly.

When you are satisfied with the points above please complete the daily checks sheet provided before putting the system into use.

**Ramblegard Technical Support Helpline:
028 9448 3979**

