

**Product Name:**  
Jack Multi Adaptor

**Product Code:**  
RG 4

**Description:**

The 'Jack' universal adapter enables you to attach more than one item to a nurse call socket. You **must** have a call lead plugged in at all times or the unit will alarm continuously. This safety feature alerts staff when a call lead or any other accessory is removed.

**Connection Points**

- a. Mono/Stereo switch - To identify how your system is set up, check the number of black rings on an existing plug (Mono = 1 Ring, Stereo = 2 Rings)  
*Please note: The Jack adaptor plug will always have 2 rings as it is interchangeable.*
- b. Sensor Mat socket. Attach lead connecting to the monitor(Bedgard/Seatgard) or lead connecting to sensor mat(Floorgard).
- c. Call lead socket. This must have an accessory(call lead) plugged in at all times.
- d. Port 2 socket to facilitate additional accessory option.
- e. Port 2 on/off switch **must** be in the off position when not in use.

The adaptor is now configured for your system.



**For use with:**  
All Nurse Call (NC) options



Mission Statement

“To assist carers **stay a step ahead** in monitoring patients, with primary focus on prevention of falls.”

**Ramblegard**<sup>®</sup>  
stay a step ahead