

Follow the steps below to set-up your fall prevention equipment:

1.

TEST SET:

- a) Move switch to **VIB+TONE**
- b) Walk in front of motion sensor on wall in W.C. The pager should then beep and vibrate



2.

PREPARE:

- a) Ensure pager is switched to **OFF** position
- b) Ensure pager is mounted on wall bracket outside of W.C.



3.

TAKE PATIENT INTO W.C.

- a) Ensure motion sensor is correctly angled down in front of W.C./patient.
- b) Leave the patient in private closing the door behind you.



4.

PAGER

- a) Move switch to **VIB+TONE**
- ** SENSOR NOW ACTIVE**



5.

PAGER

- a) Pager will alarm when patient moves.
- b) Switch pager **OFF** and attend to patient.

