

# Quick Start Guide

## Wireless Motiongard PIR System

**Applies to product codes:**

R16-0004 | R16-0004I | R16-0005 | R16-0006

System Contents	
Monitor	RG44 Wireless Motiongard PIR
Sensor	RG2 Wireless Companion Monitor RG5 Wireless Jack RG41 Wireless Jill RG43 Wall Mount Display

### **Sensor Placement**

**APPLICATION NEXT TO THE BED:** Wall mount the sensor vertically next to the headboard angled downward down the side of the bed to monitor when a patient leaves the side of the bed.

**APPLICATION ABOVE PATIENT HEAD:** Wall mount the sensor horizontally above the headboard, angled slightly downward towards the footboard, to monitor when a patient sits up in bed.

**APPLICATION ON THE SIDE CABINET:** Place the sensor on a bedside table, vertically oriented to monitor when a patient leaves the bed.


**APPLICATION ON THE FLOOR:** Place the sensor on the floor, horizontally configured to detect someone stepping out of the bed onto the floor.

### **Receiver/Monitor Placement**

The receiver item should be placed out of the field of view of the sensor, to be able to reset alarms without re-triggering the sensor.

### **General Operation**

The motion sensor will pick up movement of people in front of it and trigger an alarm, which can be reset.

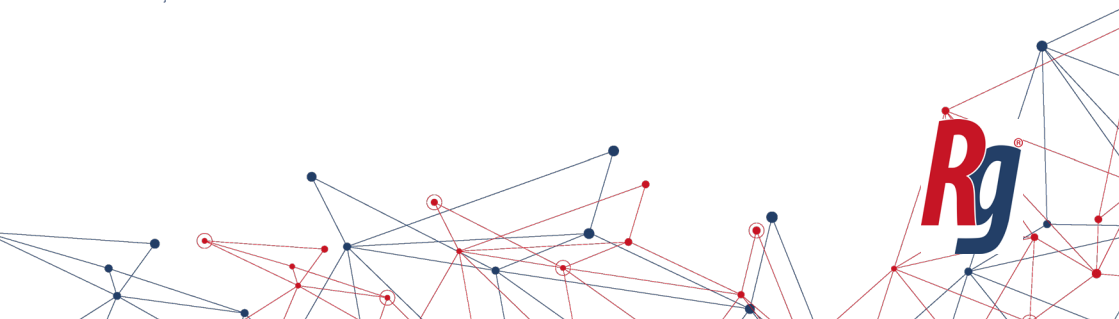

# Quick Start Guide

## Wireless Motiongard PIR System

Follow the steps below to set up your equipment:

- 1.** Pull out the battery tab to power up the wireless motion sensor or insert 2 AA batteries into the battery compartment. To do this, unscrew the bottom of the unit, then unclip and remove the cover.
- 2.** Fix receiver in the desired location using the supplied hardware.  
*Pager - lanyard or wall mount bracket*  
*Companion - hanging strap or wall mount bracket*  
*Display Panel - wall mount bracket or desk mounted*  
*Jack Fly - wall mount bracket*  
*Jack - no hardware needed*
- 3.** Ensure that the sensor pad is placed in your desired location and powered on.
- 4.** To test, activate the sensor by walking within its field of vision to confirm that it is alarming and resetting correctly.
- 5.** For resetting...  
**with Wireless Companion monitor** - press reset on the monitor and alarm will clear.  
**with Wireless Jack** - once it is alarming, triggering your nurse-call, reset at your nurse-call point.  
**with Wireless Jill** - press and hold the cancel button.  
**with Wireless Display Panel** - press and hold reset button on front of device.

*When you are satisfied with the points above please complete the daily checks sheet provided before putting the system into use.*

**Ramblegard Technical Support Helpline:  
028 9448 3979**

