


Follow the steps below to set-up your falls equipment.

1.

Check that the 'Alarm Set' numbers on each item match (if applicable)

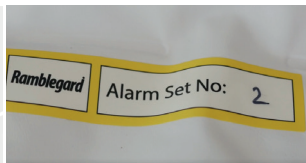

2.

Switch transmitter 'ON'

- ✓ Take transmitter out of pocket on sensor pad.
- ✓ Using a pen, move switch to 'ON' position.
- ✓ Put transmitter back into pocket and seal shut.



3.

Insert 3 x AA batteries into Companion

- ✓ Replace rubber cover.



4.

Position pads & mount Companion on wall bracket

- ✓ Bed Sensor Pad:
Position at shoulders or hips
- ✓ Seat Sensor Pad:
Can be placed over or underneath cushion



5.

Test Pads

1. Apply pressure
2. Listen for two beeps and/or FLASH
3. Release pressure
4. Listen for alarm tone and/or FLASH

